

Cetilar[®] Tape

PHARMANUTRA


Inelastic adhesive strip

for joints, muscles, tendons, based on cetylated fatty acids (CFAs)

INDICATIONS: Cetilar[®] Tape is an inelastic adhesive strip that, thanks to its formulation based on cetylated fatty acids (CFAs) of vegetable origin, is indicated for the treatment of joint disorders caused by osteoarthritis, to improve the joint function and movement ability and to reduce painful symptoms. It rapidly relieves joint and musculoskeletal pain; it is useful for functional recovery during rehabilitation after joint inflammation or injuries; it is useful in case of sport injuries to the joints to recover the normal motility.


HOW TO USE:

PUT OUT

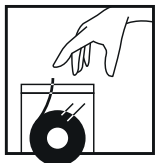


1. Open the package and take out the strip.

CUT

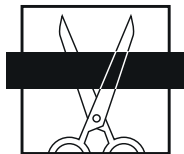


2. Cut the strip to the required size.

APPLY

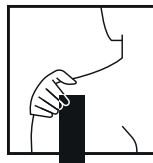

3. Make sure the skin is clean and dry before applying. Press lightly onto skin and ensure that the edges are sticking to the skin. Keep in situ for 8 hours.

PUT AWAY

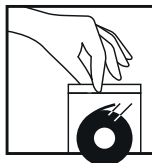

4. Put the unused product back in its packaging.

Product can be applied repeatedly throughout the day, leaving the skin exposed to air for a short period of time between applications to prevent sensitisation. The adhesive strip is intended for single use and cannot be reused.

INGREDIENTS: Cetylated fatty acids of vegetable origin, acrylate copolymer, menthol.

WARNINGS: Keep out of the reach and sight of children. For external use only. The product should only be applied to intact, clean and dry skin. Avoid contact with mucous membranes, eyes and mouth, if necessary, wash abundantly with water. Remove the adhesive strip during bathing/showering. If apparent signs of irritation or hypersensitivity to the product occur, discontinue use. Do not use in case of known hypersensitivity to the components. Do not ingest. Store in a cool, dry place, between 5°C and 40°C. Do not use the product if the packaging is damaged. Do not use after the expiry date. Do not use Cetilar[®] Tape with Menthol at the same time as other topical treatments in the area to be treated, without consulting your physician. If you are pregnant or breastfeeding, consult your physician before using the product. For use in children under 12 years of age, consult your physician before using the product.

Cetilar[®] Tape contains a proprietary blend of vegetable-derived cetylated fatty acids (CFAs) that, when applied to the painful area, helps restore mobility, improving function and mobility. The cetylated fatty acids also have an emollient action on the treated area due to their lipidic nature, which also prevents skin irritations. The menthol gives the product a fresh and pleasant fragrance.

PACKAGING: a package of Cetilar[®] Tape must contain an inelastic strip inside.



Medical device
EXTERNAL USE

PHARMANUTRA



Pharmanutra S.p.A.
Via Campodavola, 1 - 56122 Pisa (Italy)
info@pharmanutra.it - www.pharmanutra.it

CE 0373



Batch and expiry date:
see bottom of package.